LAICATM

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Please carefully read the instructions and warnings given in this handbook before use, and always keep it with the nebulizer.

The piston nebulizer is one of the most effective instruments for treating respiratory problems. The piston system gives very fast and fine nebulization, and can be used with the majority of prescribed drugs, increasing the benefits of the treatment. Thanks to the top quality performance, it could be considered a professional instrument. It is easy to use and ideal for treatment at home. Reliable, tough and lubricant free, it is made in conformity with European Standards in force regarding construction criteria and the safety of electro-medical appliances (Standard EN 60 601-1, EN13544-1).

IMPORTANT

- 1) The apparatus must only be used for nebulizing treatment. Any other use is improper and dangerous. The manufacturer is not responsible for any damage caused by improper use.
- 2) The apparatus must not be used with anaesthetics that are inflammable when mixed with air, oxygen or nitrous oxide.
- 3) Electromagnetic interference caused by the television, radio, etc., could affect the working of the apparatus, if this happens, try moving it until the interference disappears or, alternatively, plug into a different socket.
- 4) Do not use extension leads or adapters; the electric lead must be completely unwound during use to avoid dangerous overheating. Keep the lead away from hot surfaces. Never handle the plug with wet hands and never use the product while you are having a bath or a shower.
- 5) This device has to be used only with the voltage values given on the bottom of the product.
- 6) Always unplug the apparatus after use.
- 7) **Caution!** Do not intervene for any reason whatsoever on the electric cable. In case of damage, refer to a dealer.
- 8) It must not be placed in water; if this should happen, unplug it immediately. Do not attempt to touch it until it has been unplugged. Do not use the apparatus after removing it from the water (contact the retailer immediately).
- 9) Always contact the retailer for any repairs. Do not open or tamper with the apparatus. If it does not work properly turn it off and consult the instruction handbook.
- 10) Place the device on flat and stable surfaces in order to avoid occluding vent openings positioned on the sides.

11) Do not insert any object in the openings of the device.

12) Keep all the accessories out of the reach of children. Children and invalids must only use the apparatus when supervised by an adult who has carefully read the instruction handbook. Keep the bulb out of the reach of children under 3 years old, as it contains small parts that could be swallowed. Caution! This is not a toy, but a piston nebuliser for aerosol therapy. Do not leave the device unattended also when it is not used.

13) This is a medical instrument for home use, and must be used against medical prescription. It must be operated in full conformity with this instruction handbook. The patient must carefully read and understand the instructions for its use and maintenance. Contact the retailer for any further information that may be required.

OPERATING INSTRUCTIONS

- 1) Insert the plug in the power supply outlet corresponding to the appliance voltage. It is recommended to uncoil the power supply cable for all its length to avoid hazardous overheating.
 - 2) See nebulizer instructions page.
 - 3) Introduce the tube coming from the compressor OUTLET.
 - 4) Assemble the selected care accessory: mask, mouthpiece or nosepiece.
 - 5) Make sure that there is the standard supplied air filter.
 - 6) After the treatment, switch off the device and disconnect the plug. Perform a careful washing of the used parts. Before restarting the device, make sure that it has cooled and achieved ambient temperature.
- Caution:
- Always use the ampoule facing upwards in order not to let any substance or medical product come out from the ampoule itself during normal use.
 - Never inhale in horizontal position

Laica S.p.A. - Viale del lavoro, 10
36021 Barbarano Vicentino (VI) - Italy
Made in China

CE 0434 (Dir. 93/42/EEC)

ACCESSORIES SUPPLIED

- 1) Air tube
- 2) Ampoule lower body
- 3) Atomisation nozzle
- 4) Ampoule upper body
- 5) Mouthpiece

Instructions and guarantee

- 6) Mask for adults
- 7) Mask for children
- 8) Air filters

CLEANING AND DISINFECTING

Wash your hands well before beginning to clean and disinfect the accessories. Do not wash the apparatus under the tap or by soaking. If it needs cleaning, just use a cloth dampened with detergent (not abrasive and not solvent). Device with enclosure not protected against the penetration of liquids. To disinfect the accessories, it is possible to use denatured spirit or a hypochlorite solution that can be easily purchased at any chemist's shop. Before and after each use, clean all nebuliser components (exception made for the air tube) removing any medical product residuals or any dirt deposits. Clean all components with lukewarm water and/or neutral soap; rinse them carefully removing excess water and let them dry in the open air in a clean place. Never boil the accessories. To dry the humidity or condensate present in the air tube, switch on the device for 2/3 minutes letting only the tube connected; otherwise, let it dry at open air in a clean place. Clean the external part of the tube with a wet cloth.

Replacement of the spare filter: Air filter has to be replaced every 500 hours of operation or in any case when the same is rather dirty. It is suggested to replace the filter every time that the ampoule is replaced (about every 6 months). Do not wash the used filters to reuse them again. To replace the filter, lift the filter holding cover, remove the filter to be replaced and insert the spare one. Then, insert the filter holding cover in its proper seat letting it adhere well to the surface. Use only original Laica filters.

Cleaning of the ampoule: see ampoule instruction section

MICROBIC CONTAMINATION

In the presence of conditions carrying the risk of microbial infection and contamination, personal use of the accessories and nebulizer only is advised (always consult a doctor).

PROBLEMS, CAUSES AND REMEDIES

The apparatus does not turn on:

- ensure the plug is well into the socket.
- ensure the switch is in the on position (I).

The apparatus is alight but does not nebulize:

- ensure the air connection tube is not squashed or bent.
- ensure the air suction filter is not blocked or dirty. If necessary, replace with a new one.
- ensure the required drug is inside the bulb.
- make sure that the ampoule is not clogged; then, clean and disinfect the ampoule as indicated in the manual. If the washing was not successful, replace the ampoule.
- make sure that the nozzle is not badly mounted; press firmly with a finger on the nozzle (cylindrical sleeve), which is positioned within the polycarbonate ampoule bottom.

The appliance atomises slowly:

- the medicinal product may be very oily. Dilute the medicinal product using physiological saline.

The appliance is noisy after prolonged use:

- Refer to the dealer.

The appliance does not work:

- the appliance has worked close to heat sources or in environments with a temperature exceeding +40°C. The thermal protection has tripped. Let the appliance cool for at least 60 minutes and then switch on the appliance again.

N.B. if the apparatus still does not work correctly, contact the retailer.

HOW TO RETURN ITEMS

In compliance with the new European Regulations, LAICA lists here below some essential points to preserve the hygiene of appliances and operators using the same.

LAICA relies on the observance of these procedures to assure the hygiene and health of all people operating in order to obtain quality and wellness.

Each appliance that shall be returned to LAICA shall be subject to hygienic controls before technical service. In case of goods return for replacement, it is COMPULSORY to:

- disinfect carefully the external part of the main unit using a cloth moistened with denatured spirit or a hypochlorite solution.
- disinfect the accessories plunging them in the same disinfecting solutions.
- insert everything in a bag specifying "appliance with disinfected accessories".
- **always** specify the detected defect before returning the product.

TECHNICAL SPECIFICATION

Piston electrocompressor with thermal cut-out

 Class II apparatus
 Type B apparatus
 Important - please check the operating instructions
 Switch turned off
 Switch turned on
 Alternating current
 Do not use the apparatus while having a bath or shower
 Fuses: F 1 x 2A 250V
 Power feeding: 230V~ / 50Hz
 Power consumption: 140VA
 Max pressure: 250 kPa (2.5 bar)
 Max air flow: 10 l/min
 Operating pressure: 72.4 kPa (0.724 bar)
 Operating air flow: > 5 l/min at 72.4 kPa
 Neb-rate: 0.30 ml/min with 4 ml of 0.9% NaCl solution
 MMAD (measured as specifications of EN 3544-1): 1.8 µm
 GSD: 2.87
 Weight: 1.33 kg
 Dimensions: 168 x 165 x 90 mm
 Noise level: approx. 60dB (A)
 Functioning (to 40° and 110% operating voltage): 20/40 min. ON/OFF
 Min. capacity nebuliser: 1 ml
 Max. capacity nebuliser 8 ml
 Preserve in place coolness and dry land
Working conditions:
 Temperature: min. 10°C; max 40°C,
 Air humidity: min. 30% max 75%
Storage conditions:
 Temperature: min. 10°C; max 40°C,
 Air humidity: min. 30% max 75%
 Temporary use: 20/40 min. ON/OFF

DISPOSAL PROCEDURE (DIR. 2002/96/EC - WEEE)

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment. At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new

PISTON NEBULISER FOR AEROSOL THERAPY

device of the same type to be used with the same functions.

This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.

Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

WARRANTY

This equipment is covered by a 2-year warranty elapsing from the purchase date, which has to be proven by means **of the stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document.** This warranty period complies with the laws in force (Italian legislative decree No. 24 dated Feb. 02, 2002 and following "Consumers' Act" Italian legislative decree No. 206 dated Sep. 08, 2005) and applies only in case the consumer is a private entity. Laica products are designed for domestic use; therefore, their use in public businesses is forbidden. The warranty covers only production defects and is void, if the damage is caused by accidental events, wrong use, negligence or improper use of the product. Use the supplied accessories only; the use of other accessories may lead to warranty voidness. Do not open the equipment for any reason whatsoever, in case of opening or tampering of the equipment, warranty decays definitively. Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are standard supplied. Warranty decays after 2 years from the purchase date. In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com. Nothing shall be owed for the repairing or replacement of products covered by the warranty terms. In case of faults, refer to a dealer; DO NOT send anything directly to LAICA. All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product.

The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the warnings related to installation, use and maintenance of the equipment. Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

Instructions and guarantee

NEBULIZER INSTRUCTIONS

Designed and created in accordance with the latest standards, is an Medical Device (Risk Class IIa - MDD 93/42/EEC) highly efficient in medical therapies administered by means of aerosol nebulization.

SAFETY WARNINGS

- A single-patient use is recommended
- The device should be used only with compressor nebulizers for aerosoltherapy that comply with standards in force and whose use is indicated in the instruction manual
- Device not suitable for anaesthesia and lung ventilation
- The device should be used only with original accessories shown in this manual
- Information concerning connection and use with devices for aerosoltherapy are shown in the instruction manual of the devices. Always observe instructions and safety warnings as indicated in the instruction manuals of the devices for aerosoltherapy
- Use the device and its accessories as per your doctor's recommendations
- Use medications prescribed by your doctor according to his indications
- Do not use the device for other use than Nebulizer for Aerosoltherapy. Manufacturer will not be held responsible for inappropriate use of the device
- The device does not come in sterile pack; always carry out cleaning and sterilization operations before and after each treatment. When using the device follow ordinary hygiene and cleaning precautions, (wash your hands and carry out cleaning operations as described in the specific section of this manual)
- The device has small components which might be removed and easily swallowed. Use by minors and disabled people require presence of an adult with his faculties. Do not leave the device unattended in places easily accessible by minors and disabled people
- Always store the device in a dry and clean place far from light, heat and weather
- Dispose of device as provided by standards in force
- In presence of pathologies with microbial contamination and infection hazard a singlepatient use of accessories and nebulizer is recommended (always consult your doctor)

HOW TO PREPARE AND TO OPERATE THE DEVICE

The device is not sterile. Before use carry out cleaning and disinfection operations as described in the specific section. To pour the medication rotate the nebulizer top counter clockwise and remove it. Pour the quantity of medication prescribed by your doctor into the nebulizer tank. IMPORTANT: In case you do so, empty the nebulizer, clean it thoroughly and proceed with new filling. Once the medicinal product has been poured, screw again the top part on the bottom one; then connect the air tube to the air outlet hub. Connect the other end of the tube to the connection on the lower part of the nebuliser and then connect the accessories prescribed by

the doctor to the nebuliser itself.

Start the compressor for treatment to begin.
 IMPORTANT: NEVER INHALE IN HORIZONTAL POSITION.

CLEANING INSTRUCTION

Before and after each treatment clean thoroughly each component of the nebulizer removing medication residuals and possible impurities. Clean each component as described here below except the air tube which -in case of treatment to a new patient or in case of impurities should be replaced.

Clean all components with lukewarm water and/or neutral soap; rinse carefully removing excess water and let them dry in the open air in a clean place.

To disinfect the accessories, proceed as follows:

- 1) Rotate the upper part of the nebuliser counter-clockwise.
- 2) Disconnect the pisper within the bottom of the nebuliser by pulling firmly with the fingers.
- 3) To disinfect the accessories, it is possible to use denatured spirit or a hypochlorite solution that can be easily purchased at any chemist's shop
- 4) After use, disassemble the ampoule and clean all parts using lukewarm water; rinse carefully and remove excess water with a soft cloth; then, let them dry in the open air in a clean place. Never boil the accessories.

USEFUL LIFE OF THE APPLIANCE

For each single patient, it is suggested to use the ampoule for 6 months or for 120 treatments maximum. The nebuliser has to be replaced after a long standstill, if it shows deformations or breaks, or in case the atomisation nozzle is clogged by dry medicinal product, dust, etc.

Use exclusively original accessories foreseen and indicated by the Manufacturer.

STORAGE AND TRANSPORTATION CONDITIONS

Storage and transportation temperature: +10°C (50°F) ÷ +40°C (104°F)
 Storage and transportation humidity percentage: 30% ÷ 75% RH

OPERATING TECHNICAL DATA

Operating Minimum flow: > 5 l/min
 Relative operating pressure: 72.4 kPa (0.724 bar) +/- 10%.
 Min. capacity: 1 ml
 Max. capacity: 8 ml

 Read carefully instructions for use

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