**LAICA**  
TM

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PC23 - 12/2015

**LAICA**  
TM

**LAICA S.p.A.**

Viale del Lavoro, 10 - Fraz. Ponte  
36021 Barbarano Vicentino (VI) - Italy  
Tel. +39 0444.795314 - 795321  
Fax +39 0444.795324  
Made in China

[www.laica.com](http://www.laica.com)

**LAICA**  
TM

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**PC1017**

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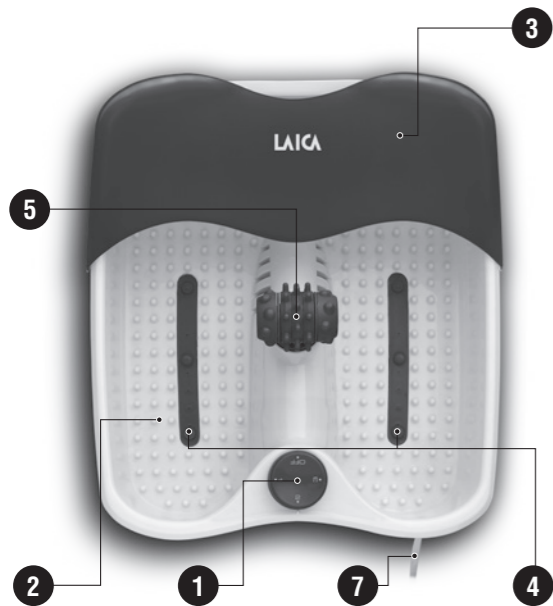
pagina 4

<b>FR</b>	Instructions et garantie <b>HYDROMASSAGE PLANTAIRE</b>	page 8
<b>GB</b>	Instructions and warranty <b>FOOT MASSAGER</b>	page 12
<b>ES</b>	Instrucciones y garantía <b>HYDROMASAJE PLANTAR</b>	página 16
<b>PT</b>	Instruções e garantia <b>HIDROMASSAGEM PLANTAR</b>	página 20
<b>DE</b>	Anweisungen und Garantie <b>UNTERWASSER FUSSMASSAGE</b>	Seite 24
<b>EL</b>	Οδηγίες και εγγύηση <b>ΥΔΡΟΜΑΣΑΖ ΠΕΛΜΑΤΟΣ</b>	σελίδα 28
<b>RO</b>	Instrucțiuni și garanție <b>APARAT DE HIDROMASAJ PLANTAR</b>	pagina 32
<b>CS</b>	Nàvod a záruka <b>VODNÍ MASÁŽ NOHOU</b>	strana 36
<b>SK</b>	Nàvod a záruka <b>MASÁŽNA VAŇA PRE NOHY</b>	strana 40
<b>HU</b>	Utasítások és garancia <b>LÁBMASZÍRÓZÓ</b>	oldal 44

**PC1017**

Fig. 1

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**FOOT MASSAGER - INSTRUCTIONS AND WARRANTY**

Dear customer, Laica would like to thank you for choosing one of our products, designed according to strict performance and quality criteria in order to ensure total satisfaction.

**IMPORTANT  
READ CAREFULLY BEFORE USE  
KEEP IN A SAFE PLACE FOR FUTURE REFERENCE**

The instruction manual must be considered an integral part of the product and must be kept for its entire life. If the appliance is transferred to another owner, its documentation must also be transferred in its entirety. To ensure the safe and correct use of the product, the user must carefully read the instructions and warnings contained in the manual insofar as they provide important information regarding safety, user and maintenance instructions.

If the instruction manual is lost or in the event you require additional information or clarifications, please contact the company at the address shown below:

Laica S.p.A. Viale del Lavoro, 10 – Fraz. Ponte – 36021 Barbarano Vicentino (VI) – Italy – Tel. +39 0444.795314 - 795321 - Fax +39 0444.795324 - www.laica.com

**INDEX**

<b>SYMBOL LEGEND</b>	<b>page 12</b>
<b>SAFETY INSTRUCTIONS</b>	<b>page 12</b>
<b>PRODUCT DESCRIPTION</b>	<b>page 13</b>
<b>Technical specifications</b>	<b>page 13</b>
<b>INSTRUCTIONS FOR USE</b>	<b>page 13</b>
<b>Preparation</b>	<b>page 13</b>
<b>Use</b>	<b>page 14</b>
<b>MAINTENANCE</b>	<b>page 14</b>
<b>TROUBLESHOOTING</b>	<b>page 14</b>
<b>DISPOSAL PROCEDURE</b>	<b>page 14</b>
<b>WARRANTY</b>	<b>page 15</b>

**SYMBOL LEGEND**

Warning		Prohibition	
1	Vibrations / Bubbles	2	Heat / Bubbles / Vibrations
3	Heat		

**SAFETY INSTRUCTIONS**

- Before using, make sure the appliance is intact and shows no visible damage. If you have any doubts, do not use the device and contact the retailer.
- Keep the plastic bag away from children: danger of suffocation.
- Before connecting the appliance, make sure the mains voltage corresponds with the voltage on the plate located on the base of the appliance.
- This device must only be used for its intended purpose and in the manner indicated in the instructions. Any other use shall be considered improper and therefore dangerous. The manufacturer cannot be considered responsible for any damage from improper or incorrect use.
- This appliance may only be used or maintained by children 8 years of age or older; individuals with reduced physical, sensory, or mental capacities or unskilled individuals under adequate adult supervision. Children must not play with the appliance.
- ⊗ NEVER leave the appliance operating unsupervised. Once the treatment is complete, turn it off and unplug it from the mains.
- In the event of a failure and/or malfunction, turn off the appliance without tampering with it. Always contact the retailer for repairs.
- ⊗ Do NOT use the appliance if it leaks.
- ⊗ Do NOT connect or disconnect the appliance with wet or damp hands or feet.
- Make sure your hands are dry when using the function selection knob.
- ⊗ Do NOT pull on the power cord or the device itself to unplug.
- Always use the selection knob to turn the appliance off and then unplug.
- Treat the hydromassage appliance with care, protect it from impacts, extreme

- temperature changes, humidity, dust, direct sunlight, and heat sources.
- ⊗ Do NOT use the device while bathing or in the shower.
- ⊗ Do NOT stand in the hydromassage and only use it when seated. Remain seated when immersing and removing feet from the water.
- ⊗ Do NOT obstruct the air intake on the bottom of the appliance. Do NOT set it on soft surfaces (e.g., bed or pillows). Do NOT insert objects in the air intake.
- ⊗ Do NOT plug or unplug the power cord when feet are in the water.
- ⊗ Do NOT use two electrical appliances at the same time for any reason (e.g., plantar hydromassage and hair dryer, plantar hydromassage and electric razor, or electric toothbrush, or mobile phone while recharging).
- Unplug immediately after use and always before cleaning and inserting or removing accessories.
- If an electrical appliance falls into water, do not attempt to extract it, but immediately remove the plug from the power socket.
- During filling and emptying, the device must always be unplugged from power.
- Only use water without adding any soaps or foaming bath salts.
- ⊗ Attention! Do NOT tamper with the power cord for any reason. If damaged, contact your retailer.
- Unwind the cable for its entire length and keep it away from heat sources.
- ⊗ Do NOT crush or bend the product and do not use it outdoors.
- After using the appliance without interruptions for 20 minutes, it must be allowed to cool for at least 20 minutes.

**⚠ ATTENTION! BEFORE USING THIS APPLIANCE**

- **In the event of health problems or for those wearing pacemakers, consult your physician before using the appliance.**
- ⊗ Do NOT use the appliance:
  - in the presence of altered skin sensation, wounds, rashes, burns, or bruises;
  - after taking medicine or sedatives or drinking alcohol;
  - if you wear a pacemaker or prosthesis;
  - if you have a severe diagnosed disease, diabetes, or vascular problems;
  - when pregnant;
  - while sleeping;

- on animals. In these cases or whenever in doubt, consult your physician before use.
- This device is not intended for professional use but only for domestic use.
- ⊗ Do NOT use the appliance in damp environments or when working with aerosol or administering oxygen.
- Before putting feet in the hydromassage, always test the water temperature.
- If the massage is not pleasant or causes pain during or after treatment, consult your physician.

**PRODUCT DESCRIPTION (see fig. 1)**

1. Function selection knob
2. Multinode bottom
3. Anti-splash cover
4. Bubbles
5. Massager with rotating nodes
6. Drain
7. Power cable

**TECHNICAL SPECIFICATIONS**

- Voltage: AC 220-240V 50-60Hz
- Absorption: 75W
- Environmental conditions for use: +10°C +40°C; RH 85%
- Environmental conditions for storage: +10°C +50°C; RH 85%

**INSTRUCTIONS FOR USE**

**PREPARATION**

- Make sure the appliance is unplugged and the function knob is in the "OFF" position.
- Fill the hydromassage with hot or cold water. The amount of water must always be between the minimum (MIN) and maximum (MAX) level indicated on the internal wall of the product.
- The appliance cannot be used to reheat cold water.

If you would like a massage with hot water, refill the appliance with hot water. The temperature will be maintained.

- Be careful not to get the function selection knob wet.
- Plug in.

 Attention! Do NOT turn the appliance on when empty, always fill with water before selecting functions.

**USE**

- 1) Sit and put feet in the hydromassager.
- 2) Select the type of function:

**1** **Vibration and Bubbles functions**

Turn the knob to position  to activate the vibrating massage and bubbles. The massage is produced by vibrations and the bubbles are produced by the movement of the water with air bubbles.

**2** **Heat, Bubbles, and Vibrations functions**

Turn the knob to position  to activate the vibrating massage, bubbles, and heat function simultaneously. The heated bottom keeps the water at a constant temperature.

**3** **Heat function**

Turn the knob to position  to activate the heat function.

- 3) The plantar hydromassage is also equipped with a rotating node massager to stimulate the soles of the feet.  
Put foot on the accessory to perform the massage.
- 4) When finished the treatment, turn the appliance off by turning the function selection knob to "OFF".
- 5) Pull the plug and empty the water using the drain located below the anti-splash cover (see fig. 2).

**MAINTENANCE**

- Clean the hydromassage with a soft, damp cloth and dry thoroughly.
- Never use chemicals or harsh products..
- Store the appliance in a cool, dry area.

**TROUBLESHOOTING**

Problem	Possible cause	Solution
The plantar hydromassage does not turn on or the functions do not activate.	• The appliance is not plugged in.	• Plug the product into the electrical socket.
	• The knob is in the "OFF" position.	• Turn on the hydromassage selecting the desired function.

**DISPOSAL PROCEDURE**

 The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment (**Dir. 2012/19/Eu-WEEE**).

At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions. If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m<sup>2</sup> without having to purchase a new, similar device.

This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.

Caution! The wrong disposal of electric and electronic equipment may involve sanctions.

 This product is meant for domestic use.

The conformity, certified by the CE-marking on the device, refers to the EEC Directive 2004/108 on electromagnetic compatibility and to the EEC Directive 2006/95 on electric material destined to be used within given voltage limits. This product is an electronic device that was tested to assure, according to the current technical knowledge, that it does not interfere with other nearby located equipment (electromagnetic compatibility) and that it is safe if used according to the advices given in the instructions for use.

In case of operation anomalies, do not keep using the device and, if necessary, refer to the manufacturer directly.

**WARRANTY**

This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means **of the stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document.**

This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products are designed for domestic use only; therefore, their use for commercial purposes is forbidden.

The warranty covers only production defects and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product.

Use the supplied accessories only; the use of other accessories invalidate the warranty cover. Do not open the equipment for any reason whatsoever; the opening or the tampering of the equipment, definitely invalidate the warranty.

Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are included.

Warranty decays after 2 years from the purchase date. In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com. Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything directly to LAICA.

All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product.

The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment.

Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

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