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LAICA TERMOMETRO A
INFRAROSSI FRONTALE
A DISTANZA

HC84 - 07/2018

LAICA[®]

LAICA S.p.A.

Viale del Lavoro, 10
36021 Barbarano Mossano (VI) - Italy
Phone +39 0444.795314 - 795321
Fax +39 0444.795324
Made in China

www.laica.com

LAICA[®]

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**TERMOMETRO A INFRAROSSI
FRONTALE A DISTANZA**

pagina 4

EN Instructions and warranty page 20
NON-CONTACT INFRARED FOREHEAD THERMOMETER

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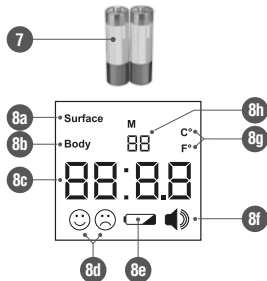
COMPONENTS

- 1) Measurement probe
- 2) "O/I" key
- 3) "SET" key
- 4) "MEMO" key
- 5) "MODE" key
- 6) Battery compartment
- 7) Batteries
- 8) LCD Display
 - a. Indication of ambient and liquid temperature measurement
 - b. Indication of forehead temperature measurement
 - c. Temperature recorded
 - d. Indication of the measurement result
 - e. Low battery indicator
 - f. Sound signal active
 - g. Unit of measure
 - h. Memory number

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**NON-CONTACT INFRARED FOREHEAD THERMOMETER - TYPE HTD8813
INSTRUCTIONS AND WARRANTY**

Dear Customer, Laica would like to thank you for choosing this product, designed according to criteria of reliability and quality for complete satisfaction.

**IMPORTANT
READ CAREFULLY BEFORE USING - KEEP IN A SAFE PLACE FOR FUTURE REFERENCE**

The instruction manual must be considered as part of the product and must be kept for its entire life cycle. If the appliance is transferred to another owner, its documentation must also be transferred in its entirety. To ensure the safe and correct use of the product, the user must carefully read the instructions and warnings contained in the manual insofar as they provide important information regarding safety, user and maintenance instructions.

In case of loss of the instruction manual or to receive further information or clarifications contact the company using the details below:

Laica S.p.A. Viale del Lavoro, 10 – 36021 Barbarano Mossano (VI) – Italy
Tel. +39 0444.795314 - 795321 - Fax: +39 0444.795324 - www.laica.com

The LAICA infrared forehead thermometer records the heat generated by the forehead to indicate the patient's body temperature. The innovative remote mode makes it ideal for measurements on children, even during sleep. It is very easy to use, allowing recording of the temperature of the body, environment and fluids. With large backlit LCD display in three different colours based on the recorded temperature.

Note: the temperature shown on the display is equivalent to the axillary temperature.

KEY TO SYMBOLS

Warning

Prohibition

CE 0598

Compliance with the Medical Directive 93/42/EEC for medical devices

Attention! Read the user instructions carefully

European representative

Symbol of "type BF applied parts"

Production batch number

Manufacturer

Serial number

IP22: Degree of protection of enclosures for electrical equipment, where the first digit indicates the degree of protection against the penetration of solid foreign bodies (from 0 to 6) and the second digit the degree of protection against the penetration of liquids (from 0 to 8).

SAFETY WARNINGS

- Before using the product, check that the appliance is intact without visible damage. If in doubt, do not use the product and contact your dealer.
- Keep the plastic bag away from children: danger of suffocation.
- This device must be used only for its intended purpose and according to the user instructions. All other types of use shall be considered inappropriate and therefore hazardous. The manufacturer shall not be deemed liable for any damage caused by inappropriate or incorrect use.
- People with physical, sensory or mental disabilities or unskilled people, may use and perform maintenance on the product only under adult supervision. Children must not play with the device.
- This appliance can be used autonomously by children of 11 years of age or older (able to read and understand the instructions for use).
- Treat the product with care, protect it from impacts, extreme variations in temperature, humidity, dust, direct sunlight and sources of heat.
- In the event of a failure and/or malfunction, turn off the device without tampering with it. Always contact the retailer for repairs.
- Ensure hands are dry when using the on and off key. NEVER submerge the product in water or any other liquids.

**ATTENTION! PRIOR TO USING THIS DEVICE**

- 1) **Self-measurement means checking, not diagnosis or treatment. Unusual values must always be discussed with your doctor. Under no circumstances should you alter the dosages of any medication prescribed by your doctor.**
- 2) This thermometer is a sensitive instrument. Handle with care and do not expose it to mechanical shocks. Do not squeeze, bend, drop or take apart the thermometer.
- 3) Keep out of the reach of children.

- 4) Do not expose the thermometer to extreme temperatures, to the direct rays of the sun and do not store it near sources of heat. Store in a dry and clean place.
- 5) Do not touch the probe with your fingers and do not dismantle the thermometer.
- 6) Do not use the forehead thermometer to measure rectal, oral or underarm temperatures.
- 7) Measurement precision can be compromised by: accentuated sweating of the forehead, the intake of vasoconstrictive drugs, skin irritations and the presence of hair or a hat on the forehead.
- 8) Clean the forehead where the probe will be pointed before each recording.
- 9) **Measure the temperature away from direct sunlight and from wind.**
- 10) This thermometer must be used to record the temperature of the forehead (at a distance of 1-5 cm).
- 11) Holding the thermometer in your hand for too long can cause a lower recording of body temperature by the probe.
- 12) Do not measure temperature for thirty minutes after physical activity or after a bath.
- 13) Do not record the forehead temperature after placing a cold pack on the patient's forehead to lower the temperature as the reading would be distorted.
- 14) The appliance may provide inaccurate readings when used in temperature or humidity conditions outside the limits indicated in the "Technical Specifications" paragraph.
- 15) Do not use near strong magnetic fields and keep away from radio systems or mobile phones (for more information on interferences see paragraph "Guidelines and declaration of the manufacturer").
- 16) When in use DO NOT touch the patient and the batteries at the same time.

SAFE USE OF BATTERIES

- Remove the batteries if the device is not used for long periods of time and keep them in a well-ventilated and dry place, at room temperature.
- ⊘ Do NOT use rechargeable batteries.
- ⊘ NEVER expose the batteries to sources of heat or direct sunlight. Failure to observe these instructions may result in damage to and/or the explosion of the batteries.
- ⊘ Do NOT throw the batteries into an open flame.
- Batteries must always be removed or replaced by adults only.
- Keep batteries out of reach of children: the ingestion of batteries can cause serious harm and death. If a battery is swallowed, seek urgent medical care.
- The acid in batteries is corrosive. Avoid contact with the skin, eyes or garments.

FEVER

Fever should be considered a symptom and not a disease. In general, it is a sign that the body is fighting an infection. In fact, it is a defence mechanism of the body characterised by an increase in body temperature above **37°C** and increased heart and breathing rates. Normal temperature within the body is about 37.5°C and this is in fact the value measured with the rectal method. The temperature measured in the mouth is about 0.5°C less (37°C) while the underarm recording is 1°C less (36.5°C). From this we can deduce the importance of a correct recording of the temperature that should be carried out by valid means, and if possible not at a time when the body temperature may be increased for physiological reasons (upon waking, after meals).

In a healthy person the temperature is affected by many factors:

- individual character of the person (individual metabolism)
- age (infants and toddlers' body temperature is higher and decreases with age) In children, the temperature varies with greater intensity, speed and frequency;
- clothing
- outside air temperature
- time of day (in the morning body temperature is lower and increases during the day)
- motor and mental activities carried out
- measurement method;
- phase of the menstrual cycle

It should be noted that the thermometer and the person must have both been in the room, in which the measurement will take place, at least 30 minutes before taking the temperature. Room temperature must be between 15°C and 40°C (59°F and 104°F).

The table below shows a list of "normal" average temperatures: **however it is recommended to familiarise yourself with your own normal temperature when you are physically well so that you can interpret the different values recorded.**

METHOD OF MEASURING AVERAGE

UNDERARM

36.5°C

ORAL

37°C

RECTAL

37.5°C

EAR

37°C

Each type of thermometer lends itself to the measurement of body temperature at a specific point of the body: the forehead thermometer only for forehead measurement; the ear thermometer only for

measurement in the ear; the digital thermometer for traditional rectal, axillary and oral measurement. Depending on the point where the measurement is made, a different temperature value is obtained. The variation can be between 0.2-1°C. You can not compare the temperatures measured with different measuring methods. It is therefore necessary to consider in case of self-diagnosis or to report to your doctor what type of thermometer and in which point of the body the temperature has been measured.

PRODUCT DESCRIPTION (see fig.1)

- 1) Measurement probe
- 2) "O/I" key
- 3) "SET" key
- 4) "MEMO" key
- 5) "MODE" key
- 6) Battery compartment
- 7) Batteries
- 8) LCD display
 - a. Indication of ambient and liquid temperature measurement
 - b. Indication of forehead temperature measurement
 - c. Temperature recorded
 - d. Indication of the measurement result
 - e. Low battery indicator
 - f. Sound signal active
 - g. Unit of measure
 - h. Memory number

TECHNICAL SPECIFICATIONS

- Product name: infrared thermometer
- Trade name: TH1003
- Classification of installation and use: internal energy, portable, continuous use, applied part of class BF, IP22, no AP or APG
- Range of body temperature measurement: 34°C to 43°C (93.2°F to 109.4°F)
- Environmental temperature and liquid measuring range: 0°C to 100°C (32°F to 212°F)
- Display resolution: 0.1°C/0.1°F
- Backlit LCD display in 3 colours (for body temperature only):

- 34 – 37.3°C/93.2-99.1°F: green, normal temperature;
- 37.4-38°C/99.3-100.4°F: yellow, fever;
- 38.1-43°C/100.6-109.4°F: red, high fever
- Measurement distance: 1 - 5 cm
- Operating mode:
 - forehead measurement ("body"): regulated mode
 - "surface" measurement: direct mode
- Nominal output range:
 - forehead measurement ("body"): 34°C-- 43°C (93.2°F-109.4°F)
 - "surface" measurement 0°C ~ 100°C (32°F-212°F)
- Batteries: 2x1.5V alkaline AAA
- Switch off: automatic, after 18 seconds of non-use
- Battery life: approximately 3000 readings (turning the thermometer on and off after each measurement)
- Operating conditions: from 15°C to 40°C (59°F at 104°F), RH ≤85%
- Atmospheric pressure: 70 - 106 kPa
- Transportation and storage conditions: from -20°C to 55°C (-4°F at 131°F), RH ≤93%
- Tolerance for body temperature:
 - from 35 to 42 °C (95-107.6°F) : +/-0.2°C (0.4°F)
 - from 34°C to 34.9°C (93.2-94.8°F) and from 42.1 to 43°C (107.8-109.4°F): +/- 0.3°C (0.5°F)
- Tolerance for ambient temperature and liquids: +/- 2°C (3.6°F)
- Clinical bias: -0,027°C
- Standard deviation: 0.14°C
- Acceptability limit: 0.26°C
- Clinical repeatability: 0.07°C

INSTRUCTIONS FOR USE

INSERTION/REPLACEMENT OF THE BATTERY

This thermometer operates with 2 x 1.5V AAA alkaline batteries. When first used, and when the display shows the battery symbol, , proceed with insertion and/or battery replacement. Open the battery compartment by sliding the cover in the direction of the arrow, then insert the batteries according to the indicated poles and close the cover. Dispose of used batteries as indicated in the paragraph "Disposal procedure".

INSTRUCTIONS FOR PROPER USE**FOREHEAD TEMPERATURE RECORDING (Body)****Attention!**

In order to ensure accurate recording of the temperature, clean the probe in its entirety with a cotton swab moistened with alcohol after each use. Make sure that the probe is completely dry before use.

IMPROPER CLEANING OF THE PROBE CAN CAUSE UNRELIABLE TEMPERATURE RECORDINGS

- 1) Clean and dry the forehead where you will be positioning the probe.
- 2) Point the probe at the centre of the forehead just above the space between the eyebrows, at a distance of approximately 1-5 cm.
- 3) Press the "O/I" key, all the function symbols (self-test) will appear on the display; immediately after the appliance emits an acoustic signal and the display immediately shows the measured value.
The green display indicates a normal temperature (between 34 and 37.3°C/93.2-99.1°F), the yellow colour indicates the presence of fever (temperature between 37.4 and 38°C/99.3-100.4°F), the red colour indicates high fever (with temperature between 38.1 and 43°C/100.6-109.4°F).
The display shows the symbol ☺ when the measured body temperature is below 37.4°C (99.3°F) and the symbol ☹ when it is higher.

When the temperature exceeds 37.4°C the thermometer emits 4 short beeps.

⚠ The temperature shown on the display is equivalent to the underarm temperature.

- 4) The appliance switches off automatically after approximately 18 seconds.
Wait at least one second between one measurement and the next.
After 5 consecutive measurements, wait at least 30 seconds before performing a new recording.

RECORDING OF THE TEMPERATURE OF THE ENVIRONMENT, SURFACES AND WARM LIQUIDS (Surface)

This thermometer allows recording of the temperature of the environment, of surfaces or of hot liquids (such as milk, water, ...).

- 1) Turn on the thermometer by pressing the "MODE" key. The display shows "37.0°C (Body)".
- 2) Press the "MODE" key again: "Surface" appears on the display.
- 3) To measure the temperature, bring the thermometer close to the relevant area (water, milk container, ...) at a distance of approximately 1-5 cm, press the "O/I" key, the appliance emits an acoustic signal and the display immediately shows the measured value.
- 4) The appliance switches off automatically after approximately 18 seconds of non-use. When it is switched on again, the display shows the forehead temperature measurement function (Body).

Wait at least one second between one measurement and the next.

After 5 consecutive measurements, wait at least 30 seconds before performing a new recording.

N.B.: Never immerse the probe in any kind of liquid.

SILENT MEASUREMENT MODE

This thermometer records the temperature in silent mode making it ideal for measurements on children even during sleep.

- 1) Turn on the thermometer by pressing the "SET" key: the display shows the forehead temperature measurement function (Body).
- 2) Press the "SET" key again: the display shows the word "OFF" and the flashing symbol 🔊 in succession.
- 3) Proceed with the measurement as described in the previous paragraphs.
- 4) To activate the acoustic signal, turn on the thermometer by pressing the "SET" key: the display shows the forehead temperature measurement function (Body).
- 5) Press the "SET" key again: the thermometer emits an acoustic signal and the display shows successively the word "On" and the flashing symbol 🔊 that remains displayed during the next measurement.

MEMORY FUNCTION

This appliance stores the last 50 measurements to assess any changes. Readings are automatically stored. Once 50 readings have been exceeded, the oldest data is automatically deleted.

To recall the stored readings, press the "MEMO" key. Each press of the key scrolls the data in the memory: number 1 refers to the most recent reading, 50 to the oldest one. The appliance switches off automatically after approximately 18 seconds of non-use.

It is possible to delete the stored data: when the appliance is on, keep the "MEMO" key pressed until the display shows "CLR", the thermometer emits an acoustic signal and the display shows the forehead temperature measurement function (Body).

THERMOMETER SETTINGS**• MEASURE UNIT - "F1" FUNCTION**

This thermometer can measure temperature in Celsius and Fahrenheit.

- 1) Turn on the thermometer by pressing the "SET" key: the display shows 37.0°C. Keep the key pressed until the display shows "F1". When the key is released, the display shows "Unit".
- 2) Press the "MEMO" or "MODE" key to select the unit of measurement and press the "SET" key to confirm it: the display shows "F2".

• "HIGH FEVER ALARM" - "F2" FUNCTION

It is possible to set the minimum temperature signalled as "high fever alarm". The thermometer is set to signal this

alarm, red display, starting from 38.1°C.

- When the display shows "F2" releasing the "SET" key, 38.1°C flashes on the display, increase the set temperature with the "MEMO" key or decrease it with the "MODE" key. Confirm the data by pressing the "SET" key; the display shows "F3".

• SILENT MEASUREMENT MODE - "F3" FUNCTION

It is possible to set the silent measurement mode:

- When the display shows "F3", when the "SET" key is released the display shows the word "On" and the symbol  flashing. Press the "MEMO" or "MODE" key to remove the acoustic signal; the display shows "Off" and the symbol  flashing. Confirm by pressing the "SET" key, the display shows "F4".

• "F4" FUNCTION

To meet the different characteristics of the world population or the different environmental conditions, it is possible to increase or decrease the measurement parameters of the thermometer.

- When the display shows "F4" when the "SET" key is released, the display shows 0.0°C.
- Increase the data by 0.1°C by pressing the "MEMO" key or decrease it by 0.1°C by pressing the "MODE" key. Confirm the data by pressing the "SET" key, the display goes out.

 The default thermometer setting is 0.0°C, the parameter adjustment range is -5°C to + 5°C.

MAINTENANCE

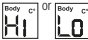
Before and/or after each use, clean the probe of the thermometer with a cotton swab moistened with alcohol to ensure accurate readings. The probe is the most delicate part of the thermometer, pay particular attention to it during cleaning.

Allow the probe to dry completely for at least 30 minutes. The body of the thermometer should be cleaned with a soft, dry cloth. THE BODY OF THE THERMOMETER MUST NOT BE WASHED WITH WATER.

Never use abrasive products and do not immerse the thermometer in water or other liquids.

Keep the thermometer in a clean, dry place, away from direct sunlight.

TROUBLESHOOTING

Problem	Possible cause	Solution
The device does not turn on.	The batteries are worn.	Replace the batteries.
	The batteries have not been installed correctly.	Check the polarity of the batteries during insertion.
The display shows 	The batteries are flat.	Replace the batteries.
The display shows 	The measured body temperature does not fall within the measurement range of the thermometer (34°C and 43°C - 93.2°F and 109.4°F).	When measuring the temperature of the environment, of surfaces or liquids, "surface" measurement must be selected. Reread the paragraph "Measurement of the temperature of the environment, of surfaces and of hot liquids (surface)".
The display shows 	The measuring distance is greater than 5 cm.	Bring the thermometer closer to the forehead during measurement.
	The forehead is covered by hair or a hat.	Uncover the forehead during the measurement.
	Cold compresses have been applied on the forehead to lower the temperature.	Do not measure immediately after applying cold compresses.
	The measurement parameters of the thermometer ("F4" Setting) have been incorrectly set.	Set the thermometer parameters correctly. Reread the paragraph "Thermometer settings - "F4"" function.
The display shows 	The room temperature is not between 15°C and 40°C (59°F and 104°F).	Use the thermometer respecting the environmental conditions of operation (read the paragraph "Technical specifications". The thermometer must be in the room where the reading is to be taken at least 30 minutes before.

All the function symbols (self-test) flash on the display and the thermometer switches off.	The batteries are flat.	Replace the batteries. If the problem reoccurs after the batteries are replaced, the thermometer mechanism is damaged. Contact the dealer.
---	-------------------------	---

N.B. If the appliance does not resume its correct functioning despite the checks performed, contact your dealer.

DISPOSAL PROCEDURE

The symbol on the bottom of the device indicates the separated collection of electric and electronic equipment (**Dir. 2012/19/Eu-WEEE**). At the end of life of the device, do not dispose it as mixed solid municipal waste, but dispose it referring to a specific collection centre located in your area or returning it to the distributor, when buying a new device of the same type to be used with the same functions.

If the appliance to be disposed of is less than 25 cm, it can be returned to a retail location that is over 400 m² without having to purchase a new, similar device. This procedure of separated collection of electric and electronic devices is carried out forecasting a European environmental policy aiming at safeguarding, protecting and improving environment quality, as well as avoiding potential effects on human health due to the presence of hazardous substances in such equipment or to an improper use of the same or of parts of the same.

Caution! The wrong disposal of electric and electronic equipment may involve sanctions. To correctly dispose of batteries (**Dir. 2013/56/Eu**) do not throw them into household waste but rather dispose of them as special waste at specialised recycling centres.

For more information about disposal of used batteries, contact the store where you purchased the equipment containing the batteries, the municipality or the local waste disposal service.

WARRANTY

This equipment is covered by a 2-year warranty starting from the purchase date, which has to be proven by means **of the stamp or signature of the dealer and by the fiscal payment receipt that shall be kept enclosed to this document.**

This warranty period complies with the laws in force and applies only in case the consumer is an individual. Laica products are designed for domestic use only; therefore, their use for commercial

purposes is forbidden. The warranty covers only production defects and does not apply, if the damage is caused by accidental events, misuse, negligence or improper use of the product.

Use the supplied accessories only; the use of other accessories invalidate the warranty cover. Do not open the equipment for any reason whatsoever; the opening or the tampering of the equipment, definitely invalidate the warranty.

Warranty does not apply to parts subject to tear and wear along the use, as well as to batteries, when they are included. Warranty decays after 2 years from the purchase date. In this case, technical service interventions will be carried out against payment. Information on technical service interventions, under warranty terms or against payment, may be requested by writing at info@laica.com.

Repair or replacement of products covered by the warranty terms are free of charge. In case of faults, refer to the dealer; DO NOT send anything directly to LAICA.

All interventions covered by warranty (including product replacement or the replacement of any of its parts) shall not be meant to extend the duration of the original warranty period of the replaced product. The manufacturer cannot be made liable for any damages that may be caused, directly or indirectly, to people, things and pets resulting from the inobservance of the provisions contained in the suitable user manual, especially with reference to the instructions related to installation, use and maintenance of the equipment.

Being constantly involved in improving its own products, Laica reserves the right to modify its product, totally or partially, with reference to any production need, without that this involves any responsibility whatsoever by Laica or its dealers.

STANDARDS

The device complies with the following standards: EN ISO80601-2-56, EN 60601-1, EN 60601-1-2, EN 60601-1-11, EN ISO15223-1.

Manufactured by: Hetaida Technology Co., Ltd. 4F, BaiShiDa High-Tech Park, Xiang Dong Industrial Area, DaLing Shan Town, Dong Guan City, Guang Dong, China.

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Phone: +39.0444.795314 - www.laica.com

Made in China

GUIDELINES AND DECLARATION OF THE MANUFACTURER

- 1) This equipment needs to be installed and put into service in accordance with the information provided in the ACCOMPANYING DOCUMENTS;
This product needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided, and this unit can be affected by portable and mobile RF communications equipment.
- 2) Caution: Do not use a mobile phone or other devices that emit electromagnetic fields, near the unit. This may result in incorrect operation of the unit.
- 3) Caution: This unit has been thoroughly tested and inspected to assure proper performance and operation!
- 4) Caution: this machine should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, this machine should be observed to verify normal operation in the configuration in which it will be used.

Guidance and manufacture's declaration – electromagnetic emission		
The Device is intended for use in the electromagnetic environment specified below. The customer of the user of the Device should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Device use RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	The Device is suitable for use in all establishments, other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

Guidance and manufacture's declaration – electromagnetic immunity			
The Device is intended for use in the electromagnetic environment specified below. The customer or the user of Device should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±15 kV air	±6 kV contact ±15 kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/ burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Not applicable	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 sec	Not applicable	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Device requires continued operation during power mains interruptions, it is recommended that the Device be powered from an uninterruptible power supply or a battery.

Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	30A/m	30A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
--	-------	-------	---

NOTE UT is the a.c. mains voltage prior to application of the test level.

Guidance and manufacture's declaration – electromagnetic immunity

The Device is intended for use in the electromagnetic environment specified below. The customer or the user of the Device should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	Not applicable	<p>Portable and mobile RF communications equipment should be used no closer to any part of the Device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = 1,2\sqrt{P}$ $d = 1,2\sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2,3\sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>
Radiated RF IEC 61000-4-3	10V/m 80 MHz to 2.5 GHz	10V/m	

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Device is used exceeds the applicable RF compliance level above, the Device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Device.
- b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the Device.

The Device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 Hz to 2.5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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Wellkang Ltd, Suite B, 29 Harley Street LONDON W1G 9QR, England, United Kingdom
Distribué par: Laica S.p.A.